



AMERICAN  
KENNEL CLUB<sup>SM</sup>

**COMPANION ANIMAL RECOVERY**

## Multi-use Microchip Implantation Device



- Remove lancet containing microchip from sterile packet and slide onto front of implanter.
- Align wings vertically and keep pressure on lower wing with index finger during entire implantation process.
- Remove the plastic cap while keeping pressure on lower wing.
- Align implanter so that the lancet is oriented bevel side up.
- Compress handle completely to insert microchip.
- If you have any questions regarding the use of the implanter, please call **1-800-252-7894**.

AKC Companion Animal Recovery  
8051 Arco Corporate Drive, Suite 200, Raleigh, NC 27617-3900