


Using The AKC CAR ProScan 700 Multi-System Scanner

1. Remove the plastic wrapping and insert batteries prior to use.
2. Press and release the yellow button to turn the scanner on. The scanner will light up and indicate the number of the last microchip read, if applicable.
3. **Please note: press & hold the Yellow button down** continuously while scanning the pet. (This applies to steps 4 through 7.)
4. Hold the ring of the scanner at a **30° angle** against the pet with the right or left edge touching the pet. The right and left edges of the ring are the optimal read areas for the scanner.
5. Scan the area between the shoulder blades for optimal microchip detection.
6. Scan slowly, giving the scanner time to detect the microchip. If no microchip is detected between the shoulder blades, scan a broader area, including the front legs down to the elbows and the neck.
7. When a microchip is detected, the scanner will give a high-pitched beep and the screen will display the type of microchip along with the unique microchip number.
8. **The scanner will automatically turn itself off after approximately 30 seconds of inactivity.**
9. For future reference: if the scanner is working slowly, replace the four AA batteries.
10. Also included in the box is a USB cable for future upgrades to the scanner.

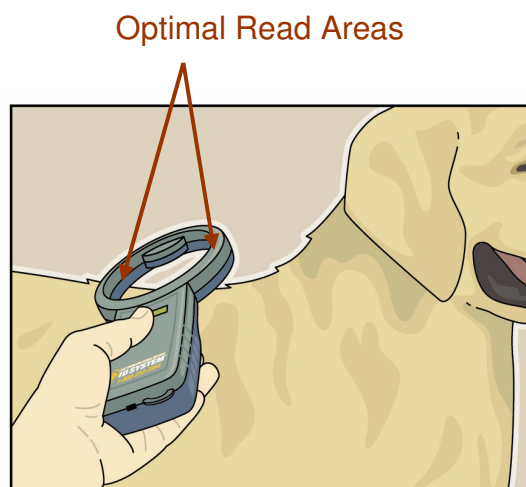

AMERICAN
KENNEL CLUBSM

COMPANION ANIMAL RECOVERY