


USING THE AKC CAR MULTI-SYSTEM POCKET SCANNER

1. Remove the plastic wrapping around battery prior to use.
2. Press and release the red button to turn the scanner on.
The scanner will sound an audible beep, and the screen will light up and indicate the # of the last chip read.
3. **PLEASE NOTE: PRESS & HOLD THE RED BUTTON DOWN** continuously while scanning the pet. (This applies to steps 4 through 7)

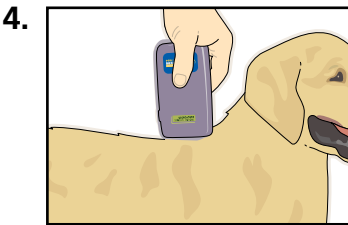

Hold the scanner against the pet with the red "scanning" end pointed down toward the pet.

5. Orient the scanner parallel to the spine between the shoulder blades when scanning for optimal microchip detection.
6. Scan slowly, giving the scanner time to detect the microchip.
7. When a chip is detected the scanner will beep and the screen will display the brand of microchip along with the unique microchip number. If it is a 134 kHz ISO chip, it will read "chip detected."
8. **THE SCANNER WILL AUTOMATICALLY TURN ITSELF OFF AFTER APPROXIMATELY 30 SECONDS OF INACTIVITY.**
9. For future reference: if the scanner is working slowly, replace the battery (9 volt).