

AKC CAR ID-1000 Solo Plus Instructions

Microchips should be administered by or under the direct supervision of a veterinarian and in accordance with any state and local statutes and regulations applicable to microchips and their implantation.

1. Scan the animal to find out if it already has a microchip. If it does, call AKC CAR immediately at **1-800-252-7894**.
2. Scan the lancet in the package to confirm the ID number of the chip is the same as the one on the barcode label in the package.
3. Remove the implanter from the sterile packet.
4. Place index and middle fingers on wings with thumb on the end.
5. Remove the plastic cap.
6. Locate the injection site (between the shoulder blades), and using the thumb and index finger, pull some loose skin to tent the skin.
7. Align implanter so that the lancet is oriented bevel side up.
8. Insert the lancet at the base of the tent at a shallow angle. Insert the lancet sub-cutaneously up to the hub to ensure contact with the tissue.
9. To insert microchip, compress handle completely until it clicks.
10. Remove the lancet from the animal. Release tented skin and apply pressure to the injection site.
11. Replace the protective cap on the used lancet.
12. Scan the injection site to ensure the microchip is working correctly and double-check the number on the packaging.
13. For immediate enrollment, go to www.akccar.org and click on "Enroll Today." Or the pet's owner can enroll online or mail the completed paper form to AKC CAR.
14. If you have any questions regarding the use of the Solo Plus, please call **1-800-252-7894**.



Be Sure To:

- Take all necessary aseptic precautions before inserting the microchip.
- Dispose of the implanter with lancet properly after use in accordance with your local regulations.
- Refer to our terms and conditions of sale on our web site: www.akccar.org.

Thank you for choosing AKC CAR and the Solo Plus Microchip!



AMERICAN
KENNEL CLUBSM

COMPANION ANIMAL RECOVERY

Microchips. Recovery. Giving Back.

AKC CAR • 8051 Arco Corporate Drive, Suite 200 • Raleigh, NC 27617-3900

SP001 (4/09)